

# BEDSIDE ASSISTANT®



FOR HELP GETTING IN AND OUT OF BED, MORE CONVENIENT THAN A BED RAIL -  
A BED HANDLE™ MAKES ANY HOME BED A SAFER BED!

- Helps sitting, standing, rising, rolling over
- Secured between box spring and mattress,  
no need to lift mattress,  
nothing to harm sheets



**Bonus: CONVENIENT BEDSIDE POCKETS USING A STANDARD PILLOWCASE**

- 1 Pull pillowcase down over the handle, tuck in the corner points
- 2 Fold the pillowcase back over itself, pulling the hem up to the handle
- 3 Fold the pillowcase hem down again, exposing the right side of the fabric

## *Easy to install :*

- *When you are in the bed one bar should be next to your ribcage and the other bar next to your calves.*
- ***Position** bars on the same side of the bed leaving a comfortable sitting space in between.*
- ***Slide** the long ends of the bars all the way under the mattress.*
- ***Push** the handle end snugly against the side of the mattress.*
- *Bars may be pulled out a few inches to make the bed or change the sheets but must be pushed all the way in before using.*



ONE SET: MODEL BA10W

**Caution:** This equipment is designed to give you a place to put your hands for assisting while sitting, standing, rising and rolling over. Do not sit on the bed handle. Do not allow children to play with. Read other side before using. **MAY NOT BE SUITABLE FOR USE WITH SOME LIGHTWEIGHT TWIN MATTRESS BEDS. NOT FOR USE WITH HOSPITAL BEDS. TEST CAREFULLY.**

Dear Valued Customer,

Thank you for purchasing the **BEDSIDE ASSISTANT**<sup>®</sup>. We hope that this product meets your needs and makes your life a little easier. This product has been designed to assist while sitting, standing, rising and rolling over in bed. Bed Handles are like grab bars and are not bed rails. Our hope is to offer a helpful product at a reasonable price; we are not responsible for losses, damages, injuries or death caused by use or misuse thereof.

Please use the **BEDSIDE ASSISTANT**<sup>®</sup> with care and caution but use it at your own risk. Do not rely on the bed handle to support your entire weight, do not allow children to use, do not allow the handle to be obscured by bedding. If bar becomes dented or crimped discontinue use. This product may not be suitable for use with some lightweight (usually foam) twin mattress beds. Not intended for hospital beds.

### **IMPORTANT - READ CAREFULLY BEFORE USING PRODUCT**

**INTENDED USE** - This model BA10W **BEDSIDE ASSISTANT**<sup>®</sup> is designed to be used only for assisting while sitting, standing, rising and rolling over. However, *each* handle of the model BA10W **BEDSIDE ASSISTANT**<sup>®</sup> is designed to support no more than 175 pounds of weight when used. Remember, the **BEDSIDE ASSISTANT**<sup>®</sup> must only be used when each unit is completely inserted between conventional box springs and mattress with each handle portion fully inserted up to the mattress. Failure to do this will compromise the ability of the **BEDSIDE ASSISTANT**<sup>®</sup> to sustain the designed maximum weight capacity. *Avoid using this product where the bed frame may dent the **BEDSIDE ASSISTANT**<sup>®</sup>.*


**THE **BEDSIDE ASSISTANT**<sup>®</sup> OBTAINS ITS STRENGTH FROM STEEL TUBING. AS WITH ANY PRODUCT MADE FROM TUBING, ITS USE SHOULD BE DISCONTINUED IF THE TUBING SHOULD BECOME DENTED. SUCH DAMAGE MAY LEAD TO PRODUCT FAILURE AND MAY CAUSE PERSONAL INJURY. IF DENTED THE PRODUCT MUST NOT BE USED.**

**LIMITED WARRANTY** Bed Handles, Inc. warrants this product under normal use against defects in material and workmanship to the original purchaser for one (1) year from the original date of purchase. Any implied warranties, including the warranties of merchantability and fitness for a particular purpose, are limited to the duration of the express warranty described above. Some states do not allow limitations on the duration of an implied warranty, so the above limitation may not apply to you. This warranty is void if, in the sole discretion of Bed Handles, Inc., the product has been damaged by accident, neglect, misuse, abuse or other causes not arising out of defects in material or workmanship.

**DISCLAIMER OF WARRANTY** - The foregoing warranties are in lieu of all other warranties, express or implied, including but not limited to the implied warranties of merchantability and fitness for a particular purpose.

**EXCLUSIVE REMEDY** - During the warranty period Bed Handles, Inc. will repair or replace the defective unit free of charge when the unit is returned to Bed Handles, Inc. or to the dealer from which the product was purchased.

## Bedside Assistant® on a bed

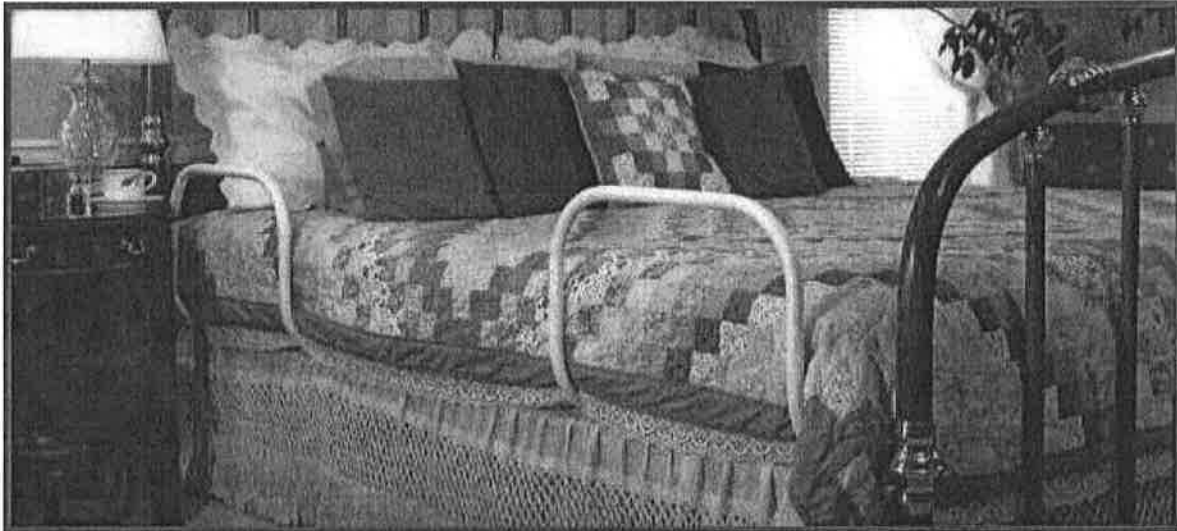

- **Convenient** - For help getting in and out of bed
- **Strong** - Each handle features one piece steel construction
- **Attractive** - White, textured, cushioned vinyl grip surface
- **Unique** - A handle for each hand, no twisting to one side
- **Economical** - Keep your own bed
- **Adaptive** - Adjustable room to sit
- **Safer** - Leaves floor space clear
- **Easy** - Super simple to install
- **Works** - with ordinary mattress & box springs, platform or water bed

[Home Page](#)

[Previous Page](#)

[Next Page](#)

Bedside Assistant (R), Travel Handles (tm) and Bed Handle (tm) and ByteSwitch (tm) are registered trademarks of Bed Handles, Inc. and are protected by one or more United States patents.



Photo # 9: View of the Caregiver demonstrating the position of the victim.

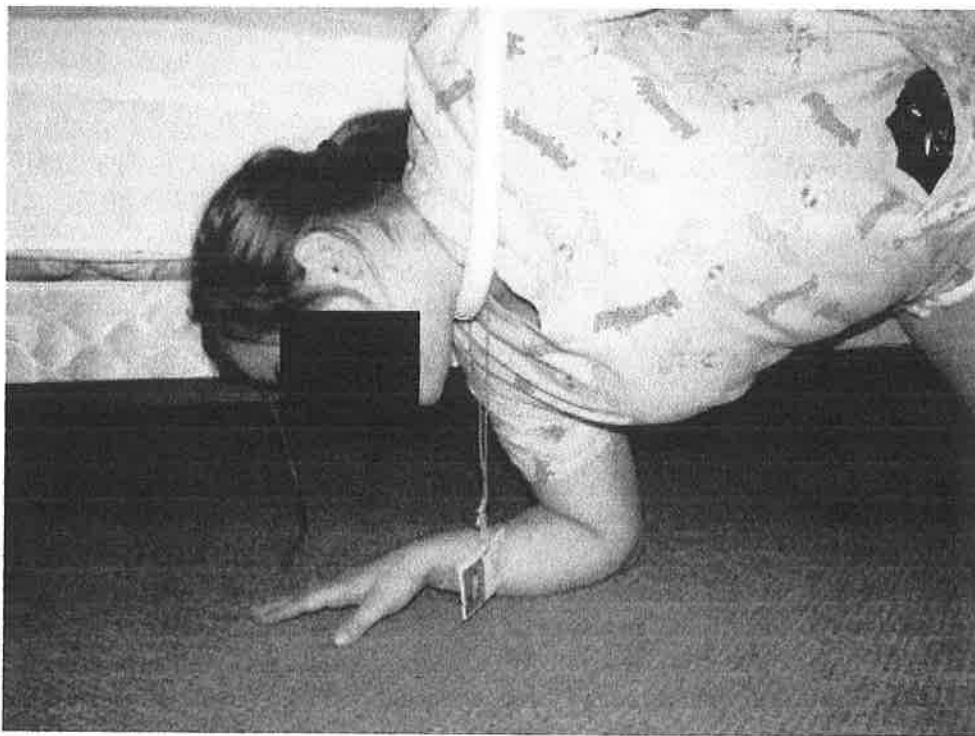


Photo # 10: Close-up view of the demonstration.